VARILITE™ Universal Cushion Repair Kit

Instruction Manual

Intended Use and Contraindications

This repair kit is intended to fix small leaks in the cushion component of VARILITE seating systems and back supports.

 **Warning:** Do not attempt to repair your cushion over an existing repair, or more than 10 times total.

CONTENTS

- (2) Fabric Cover Patches
- (1) Tube of Repair Adhesive
- (1) Wooden Applicator
- (2) Alcohol Wipes

INSTRUCTIONS FOR CUSHION REPAIR

Read through these instructions completely before attempting a repair.

What you will need to repair your VARILITE® cushion:

- A VARILITE Universal Repair Kit
- A way to mark the leak (marker or pen)
- Scissors (optional for trimming cover patches)

Locating Leaks

If the leak is not obvious, open the valve and allow the cushion to inflate as much as possible. Close the valve to prevent water from contaminating the foam. Immerse the cushion in water, small escaping bubbles will indicate a puncture. Mark the leak with a pen/marker, rinse the area and dry thoroughly before proceeding.

Repairing Your Cushion

1. Clean repair area. Lay your cushion on a flat surface with the valve open. Using an alcohol wipe (included), thoroughly clean a generous area around the puncture. Allow to dry completely.
2. Select cover patch. Ensure that the cover patch will cover the puncture and extend beyond it at least 1 3/8 in. (3.5 cm) in all directions.
3. Apply Repair Adhesive directly on the puncture. Using the wooden applicator, thoroughly work the adhesive into the fabric, achieving a thin, even layer that extends 1 in. (2.5 cm) beyond the puncture. Make sure a layer of adhesive completely covers the puncture and allow it to cure for 10-20 min., or until the adhesive is tacky.
4. Peel away backing from patch. Center the patch over the puncture and gently lay it over repair area, ensuring that the patch extends beyond the adhesive by 3/8 in. (1 cm).
5. Apply patch. Press the patch firmly with the palm of the hand to ensure the best adhesion.
6. Let cushion rest. The cushion will be ready to use after leaving it undisturbed, with the valve open, for at least 3 hours.
7. After the glue has dried, perform an additional visual inspection and dunk tank test to ensure the leak has been fixed prior to using the product.

MANUFACTURER AND WARRANTY INFORMATION

For VARILITE product warranty information, please refer to the dealer or distributor you purchased your product from.

In the event of a serious issue or injury, please report the incident to VARILITE customer service. Additionally, if you reside in the European Union, please notify the manufacturer or distributor, and the competent authority of the Member State in which you are established.

 Information printed in this manual may have changed since this document was printed. Current information is available on our website at www.VARILITE.com.

Cascade Designs, Inc.
4000 1st Ave South
Seattle, WA 98134 US
Phone: (800) 827-4548
Fax: (206) 343-5795
www.VARILITE.com

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